


BEEKLEY AccuGrid® for Superior Image Quality and Safe Specimen Handling



BENEFITS of AccuGrid :

- Alphanumeric grid provides accurate localization for the pathologist
- Sealed biohazard bag protects staff from blood borne pathogens during imaging and transport
- Low density grid does not obscure tissue detail
- Flat design requires minimal storage space
- Attached patient information tab keeps patient information permanently with the specimen

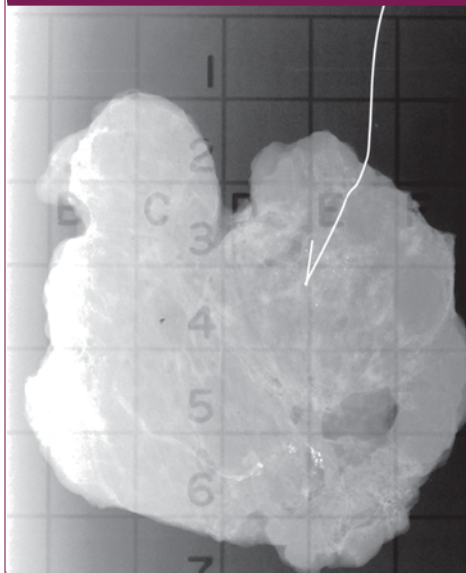

How AccuGrid is Used:

- **Surgery:**
Fill in patient information. Place the specimen into the bag, on top of the grid. Seal bag, record patient information, and transport to radiology.
- **Radiology:**
Place the AccuGrid on the mammography unit. Center the specimen on the grid by pushing with fingers through the bag (do not release seal). Apply light compression and take radiograph. Once compressed, the grid's proprietary coating holds the specimen in place so orientation to the grid is maintained. Transport to pathology with a copy of the specimen radiograph.
- **Pathology:**
Tear open the bag. Use the specimen radiograph and the grid coordinates to localize the area of concern. A pin can also be placed into the specimen to hold it in its position on the grid. Once pinned in place, all sides of the specimen can be lifted and inked without losing orientation to grid coordinates. Finally, begin gross sectioning on the grid.

Learn about other Beekley Biopsy Products by visiting www.beekley.com or calling 1-800-233-5539 to request a catalog!

Simple, disposable, one-of-a-kind biopsy solutions only from Beekley.

BEEKLEY
CORPORATION
One Prestige Lane, Bristol, CT 06010

Available in 2 sizes!

- Accugrid 4.65"
Product Code # 300
- Accugrid 5.65"
Product Code # 302

PRICING:

1-4	Cases	\$258.00/case
5-12	Cases	\$229.00/case
13+	Cases	\$200.00/case

Call 1-800-233-5539

Fax 1-800-735-1234

Order Online at www.beekley.com