

BEEKLEY



ColdSPOT[®]

Moldable freezer cold packs to meet all your cold therapy needs

Instructions for use:

1

Pre-freeze ColdSPOT for a minimum of 2 hours before use

2

Remove ColdSPOT and sterile gauze from outer packaging

3

If desired, apply gauze pad to biopsy or aspiration site

4

Remove adhesive* side tab covers

5

Apply tabs to skin centering the cold pack over the site

6

ColdSPOT can be reused by simply refreezing for another 2 hours

* ColdSPOTS are latex free and hypo-allergenic

Beekley ColdSPOTS

provide the best combination of comfort and cold for your patients

ColdSPOTS are comfortable gel packs that provide immediate relief to reduce bleeding, bruising, pain and inflammation following:

- Cyst aspiration
- Fine needle aspiration
- Biopsies of the Breast | Thyroid | Lymph node



ColdSPOT[®]

- ✓ Perfect for superficial area procedures
- ✓ Stays cold for 15-20 minutes



ColdSPOT[®] Floral

- ✓ Perfect for deeper procedures
- ✓ Advanced gel pack stays cold 30 - 45 minutes
- ✓ Features a beautiful floral design to provide emotional comfort



ColdSPOT[®] Pillows

- ✓ Perfect for deeper procedures or sensitive skin
- ✓ Advanced gel cold pack stays cold 30 - 45 minutes
- ✓ Encased in soft fabric for added patient comfort
- ✓ Light, beautiful floral pattern

Only ColdSPOT has adhesive* side tabs for exact positioning to eliminate shifting and the need for bulky bandages.

ColdSPOT®

Moldable freezer cold packs to meet all your cold therapy needs



ColdSPOT

Order Code: 500

Contents per box: 100

1-3 boxes \$2.04/each

4+ boxes \$1.73/each



ColdSPOT Floral

Order Code: 502

Contents per box: 68

1-3 boxes \$3.00/each

4+ boxes \$2.54/each



ColdSPOT Pillows

Order Code: 505

Contents per box: 50

1-3 boxes \$4.08/each

4+ boxes \$3.46/each

ColdSPOT pricing: 1-3 boxes \$204.00 per box | 4 + boxes \$ 173.00 per box

Combine quantities of all types of Beekley ColdSPOT products to earn maximum discounts

**To place an order, call 1-800-233-5539,
fax 1-800-735-1234 or online at www.beekley.com**

BEEKLEY

One Prestige Lane, Bristol, CT 06010 Tel: 1-800-233-5539 Fax: 1-800-735-1234 www.beekley.com
0909