


Beekley Cradles™ Application Instructions

Cradles shown on paper backing with oval foam tab.

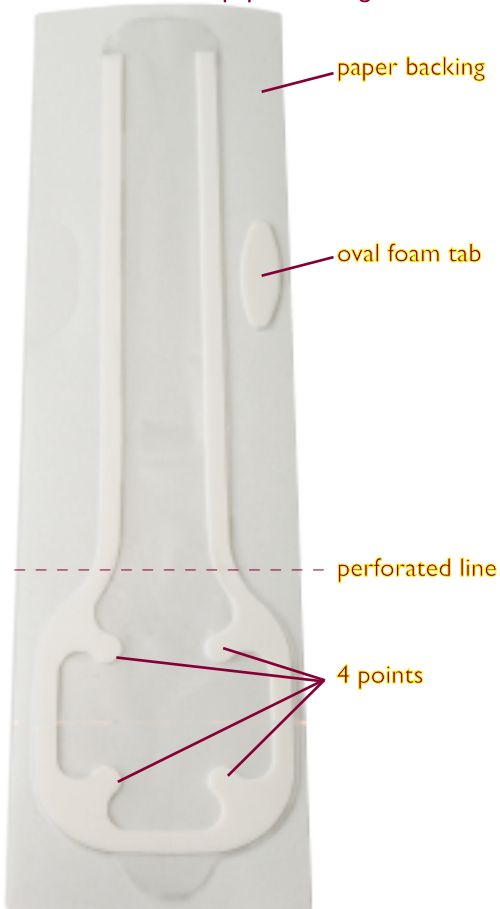

To start using
Cradles™

today contact your
**Breast Biopsy
Sales Specialist at:**

1-800-233-5539

fax: 1-800-735-1234

e-mail us at Cradles@beekley.com

online at www.beekley.com




1. Tack down end or cover tip of wire with **oval foam tab** (optional).



4. Remove remaining **paper backing** and position long strip over wire. Press to apply foam adhesive border



2. Expose square end first by pulling paper back to the **perforated line**. Place enclosed sterile gauze over **4 points** under Cradles or over wire exit site on the patient.

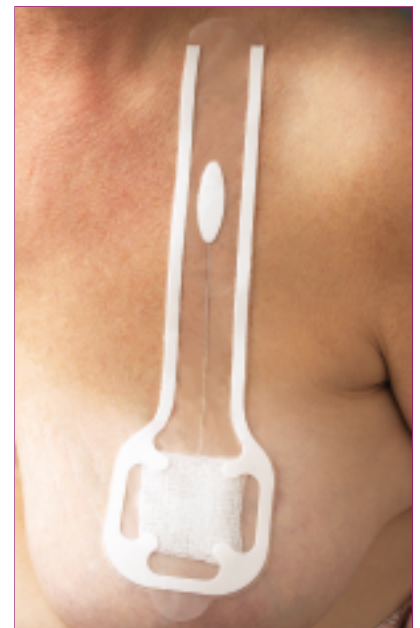

5. When removing the Cradles, pull either end tab



3. Position and adhere square end over the wire exit site.

Note: If Cradles are too long, before removing paper backing, cut excess strip to appropriate length. The half-moon shaped tab can be popped out of the paper liner and applied to the cut end to replace the pick-up tab for removal.