


## **GuideLines®** **Instructions For Use:**

---

---

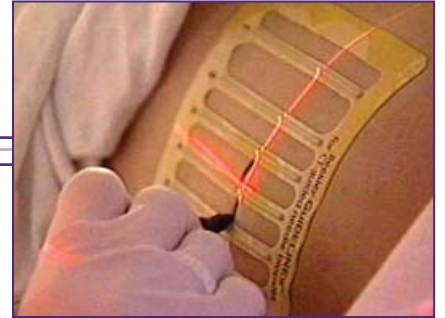

- 1** Take an initial pilot scan to identify the area of interest.
  - 2** Remove the backing from GuideLines to expose the adhesive.
  - 3** Place GuideLines on the patient, directly over the area of interest.
  - 4** Scan the area of interest and determine which slice the radiologist will want to work from.
  - 5** Using the series of lines on the scan, determine the preferred area for needle entry.
  - 6** Move the patient to the predetermined scan location.
  - 7** Using the open windows between the lines on GuideLines mark the needle entry site based on where the laser shines.
  - 8** If the preferred point of entry falls on a grid line simply mark on both sides of the line.
  - 9** Remove GuideLines, prepare the patient and perform the biopsy.
- 
- 

**To start marking with GuideLines®  
today, contact your Beekley representative:**

**Tel: 1-800-BEEKLEY (233-5539) Fax: 1-800-735-1234 Web: [www.beekley.com](http://www.beekley.com)**