

GUIDELINES[®]

CT grid for localizing needle entry sites

Instructions for Use:

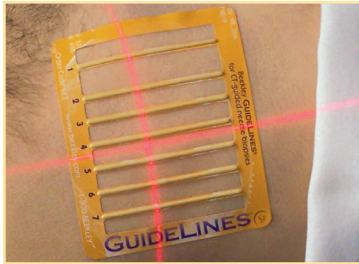


Fig. 1

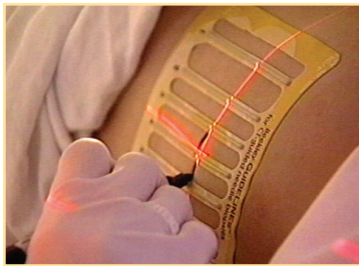


Fig. 2

1. Remove backing from GuideLines CT biopsy grid to expose adhesive.
2. Place GuideLines on patient over area of interest.
3. Using the series of lines on the scan, determine preferred area for needle entry. (Fig. 1)
4. Using open windows between the lines, mark needle entry site. If the preferred point of entry falls on a grid line, simply mark on both sides of the line. (Fig. 2)
5. Remove after use.

REF 217 QTY 15 / box

REF 117 QTY 20 / box



Manufactured by Beekley Corporation
One Prestige Lane, Bristol, CT 06010 USA
Tel: 1-800-233-5539 or +1-860-583-4700 Fax: 1-800-735-1234
www.beekley.com

BEEKLEY, BEEKLEY MEDICAL, GUIDELINES, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. BEEKLEY is a Registered Community Trademark. BEEKLEY is a registered trademark in Canada.

© 2019 Beekley Corporation. All rights reserved.

WARNINGS AND PRECAUTIONS:

- Single Patient Use Only;
- Do Not Reuse 
- Latex-Free
- Lead-Free
- Non-Sterile 



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

REV: GL_IFU-WEB_0219