

Fig. 1



Fig. 2




Fig. 3



Fig. 4



Fig. 5

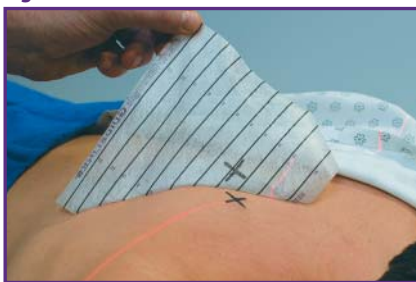

Instructions for Use:

1. **Remove grid from backing to expose adhesive.** (Fig. 1)
2. **Place grid on patient directly over area of interest.** (Fig. 2)
3. **Smooth grid onto patient's skin to ensure complete adhesion and eliminate any air bubbles that may occur.**
4. **After scanning, use the series of 11 dots on the image to determine needle entry site.** (Fig. 3)
5. **With a fine point Sharpie[®] marker, make mark where laser crosses the appropriate line on the grid. Ink instantly transfers through the grid material to mark entry site on skin.** (Fig. 4)
6. **Remove grid from patient and discard.** (Fig. 5)





Manufactured by Beeley Corporation
One Prestige Lane, Bristol, CT 06010 USA
Tel: 1-800-233-5539 or +1-860-583-4700 Fax: 1-800-735-1234 www.beeley.com

BEEKLEY, BEEKLEY MEDICAL, GUIDELINES, MARK THROUGH GUIDELINES, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. BEEKLEY is a Registered Community Trademark. BEEKLEY is a registered trademark in Canada. © 2017 Beeley Corporation. All rights reserved.

REF 219

WARNINGS AND PRECAUTIONS:

- Non-Sterile 
- Single Patient Use Only; 
- Do Not Reuse
- Hypoallergenic
- Latex-Free
- Lead-Free

CE

EC REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

REV: MTGL_IFU_0617